




EYE THERAPY

SETTINGS:

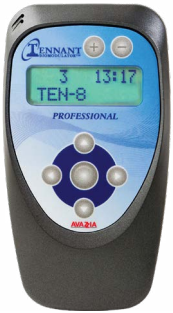

FOR PRO

Set Mode to
VA-Step.

Set Power to
250.

OR



FOR PLUS

Set Mode to
Infinity.

Set Power to
solid orange dot.



POSITION:

Hold the BioTransducer®
Crystal Wave directly against
the temples (as shown).



DURATION:



Treat **both sides** for
15 minutes each.



EYE THERAPY

TREATING EYES WITH THE BIOTRANSDUCER® PRO

SETTINGS:



FOR PRO

Set Mode to
VA-Step.

Set Power to
250.

OR



FOR PLUS

Set Mode to
Infinity.

Set Power to
solid orange dot.



POSITION:

Hold the BioTransducer® Pro at the temple, shining the light through
the side of the eye



DURATION:

Treat **one side** for for 5 minutes.

*BT Pro only requires treating one side, but you may treat both if you desire.
Only treat eyes for a maxium of 10 minutes twice a day.*

Senergy Medical Group, 9901 Valley Ranch Parkway East, Ste 1009, Irving TX 75063
(972) 580-0545 • (866) 514-8221 • be.well@senergy.us